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## en INSTRUCTIONS FOR USE

Orthodontic appliance acrylic, cold cure polymerisable based on methyl methacrylate in powder and liquid form. Free of cadmium, phthalates and BPA.

## 1. Indications

For the fabrication and repair of orthodontic appliances using the dough technique or the spray-on/salt-and-pepper technique. Classification as per DIN EN ISO 20795-2 type I and as per Medical Devices Directive 93/42/EEC Annex IX class IIa for removable and fixed orthodontic appliances. Only for intended use by qualified personnel (dental laboratory/dental practice).

## 2. Contraindication

Patient contact with the unpolymerised acrylic or the individual components is contraindicated. With a confirmed allergy to one or more of the components of the medical device, the surface must be once again wetted with the liquid and smoothed.

## 3. Properties

Good stability, universal application, comfortable range of processing options, reliable and rapid curing, tested and certified biocompatibility.

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## 5. Processing instructions

## Preparation

The plaster model must be wetted until no more bubbles escape from the plaster so that during pressure polymerisation and when allowing the acrylic to flow or when pressing, no air is forced from the plaster into the acrylic to form bubbles.

## 6. Recommendation after completion

Store the appliance in water until it is incorporated. As a general rule, 24-hours' storage in water is recommended to achieve further breakdown of the residual monomer content and thus prevent any taste disturbances.

## 7. Storage

Do not use the material after the expiry date has passed. Do not store above 25°C. Keep out of direct sunlight. Always seal the container well after use. Keep away from heat/sparks/open flames/hot surfaces. Don't smoke.

## 8. Disposal

Dispose of in accordance with official regulations. Do not dispose of individual components in household waste or the sewage system. Completely polymerised plastic, however, can be disposed of in household waste.

## 9. Feedback:

Please quote batch number and article number in all correspondence about the product. According to the EU Medical Device Directive, users/patients are obliged to report serious incidents related to a medical device to the manufacturer and the national competent authorities in the country in which they occurred.

## Spray-on/salt-and-pepper technique

When the alginate insulation is thoroughly dried, the monomer is applied to the plaster model either from a dosing bottle or using a brush and the model is then immediately sprayed with the polymer.

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## 5. Processing instructions

## Preparation

\*The times stated refer to a material and room temperature of 23°C. Higher temperatures shorten these times while lower temperatures extend them.

## 6. Recommendation after completion

The model in plaster must be imbibed until no more bubbles escape from the plaster so that during pressure polymerisation and when allowing the acrylic to flow or when pressing, no air is forced from the plaster into the acrylic to form bubbles.

## 7. Storage

Store the appliance in water until it is incorporated. As a general rule, 24-hours' storage in water is recommended to achieve further breakdown of the residual monomer content and thus prevent any taste disturbances.

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