

1. Indication

Consumable material for sandblasting-equipment in the dental laboratory.
Application includes condense, smoothing and pre-polishing metal surfaces without material removal.

2. Application

We recommend a blasting pressure of approx. 2 bar, which must be adapted by the dental technician according to the situation. Direction should be perpendicular or in a pointed angle to the surface at a distance of approx. 1 cm!

3. Technische Werte

Lead free natron glass beads

Grain size:

125 μ for non-precious alloys

50 μ for precious, semi and non-precious alloys

4. Recommendations and hints

Also observe recommendations and instructions of the blasting equipment manufacturer.

Never use blasting equipment without suitable suction and appropriate personal protection. (Glasses, mask and gloves)

Never last toward the eyes or uncovered skin!

5. Shelf life

At least 24 months in sealed containers in a dry environment.

Blasting agents are hygroscopic and must be protected from moisture.

6. Warranty

SHERA Werkstoff-Technologie GmbH & Co. KG is certified according to DIN EN ISO13485 and due to a thorough quality control system guarantees, a flawless quality of its products. Our instructions for use are based on the results of our test laboratory. The technical data given can only be guaranteed if the processing is carried out as mentioned. The user is self-responsible for processing of the products. We are not liable for faulty results as SHREA has no influence on the processing. Nevertheless, possibly arising claims for damage relate only to the value of the product.