

SHERASON 928

INSTRUCTIONS FOR USE

1. Indication

SHERASON 928 is a special cleaner for removing plaster from dental workpieces and aids, such as crown stumps, impression trays, instruments or model plates and cuvettes.

2. Contraindication

Not suitable for cleaning sensitive metals or non-resistant alloys. Test on a suitable area first. Not for large-area cleaning; please observe the safety instructions.

3. Safety instructions

Causes skin irritation.
Causes serious eye damage

Do not breathe mist/vapour/aerosol.
Wear protective gloves/eye protection.
IF ON SKIN: Wash with plenty of water.
IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
If eye irritation persists: seek medical advice/attention.

SHERASON 928 may only be used in sealable containers made of glass, stainless steel or suitable plastic (e.g. PP, PE).
Improper use may release vapours.

4. Processing instructions

Preparation

Place workpieces in a suitable container with a lid. For sensitive materials, check material compatibility beforehand.

Cleaning time

The cleaning time depends on the amount of plaster and the degree of moisture. The following information is a guideline:

- Ultrasonic bath: approx. 2–8 minutes
- Manual cleaning: up to 60 minutes
- Repeat the process if necessary

After cleaning, rinse the workpieces thoroughly under running water

5. Storage

Store in the original container in a well-ventilated place.

6. Disposal

Small quantities can be disposed of in wastewater after being heavily diluted.
Do not dispose of larger quantities together with household waste.
Do not allow to enter the sewage system.

7. Warranty

SHERA Werkstoff-Technologie GmbH is certified according to EN ISO13485 and guarantees for the products, due to a thorough quality control system, a flawless quality of its products. Our instructions for use are based on the results of our test laboratory. The technical data given can only be guaranteed if the processing is carried out as mentioned. The user is self-responsible for processing of the products. We are not liable for faulty results as SHERA has no influence on the processing. Nevertheless, possibly arising claims for damages relate to the value of the products only.



SHERA Werkstoff-Technologie GmbH

Espohlstr. 53 | 49448 Lemförde | Germany

Tel: +49 (0) 5443-9933-0 | www.shera.de