



# Cleaning and care instructions

for SHERAprint-splint taff & flex



# Cleaning and care instructions

Dear patient,

To ensure that you can enjoy your splint, repositioners, mouthguard or nightguard for a long time, we recommend:

## Daily cleaning

Use lukewarm water, a soft toothbrush or liquid soap for daily cleaning. Rinse well afterwards.

## Advice

- Do not use toothpaste, the abrasive particles are coarse and scratch the surface.
- Do not use acids or solvents.
- Do not use mouthwash



**SHERA** Werkstoff-Technologie GmbH

Espohlstr. 53 | 49448 Lemförde | Germany  
Tel.: +49 (0) 5443-9933-0 | [www.shera.de](http://www.shera.de)