

Indication

Scanspray for extraoral CAD/CAM use. Reduces the surface reflection thus achieving an ideal scan of the models.

Scope of delivery

- 200 ml spray can,
- 1 spray head with angled nozzle for straight to a point application,
- 1 spray head with straight nozzle for application on surfaces.

3. Application

- The surface to be sprayed should be clean and dry.
- Put the spray head on.
- Shake can well before using.
- Hold can vertical while spraying.
- Press spray head always well, thus avoiding plugging of the nozzle.
- Apply a thin spray layer on the corresponding surface from a distance of approx. 5 to 8 cm.
- A thin and complete wetting is achieved by spraying as short as possible.
- After drying of the surface the scan can get started.
- After recording the sprayed surface can be cleaned with water or steam.

Security advice

Container is under pressure. Protect against sunlight and temperature of more than 50°C. Even after use do not open by force nor burn. Do not spray into open flames or smouldering objects. Take away from sources of ignition – do not smoke. Keep out of reach of children.

Warranty

SHERA Werkstoff-Technologie GmbH & Co. KG is certified according to DIN EN ISO13485 and guarantees for the products, due to a thorough quality control system, a flawless quality of its products. Our instructions for use are based on the results of our test laboratory. The technical data given can only be guaranteed if the processing is carried out as mentioned. The user is self-responsible for processing of the products. We are not liable for faulty results as SHREA has no influence on the processing. Nevertheless possibly arising claims for damages relate to the value of the products only.

